



Automated **E**xternal **D**efibrillator

Verify unconsciousness





**If no response, call or have
someone call**

911


Begin CPR



1



2

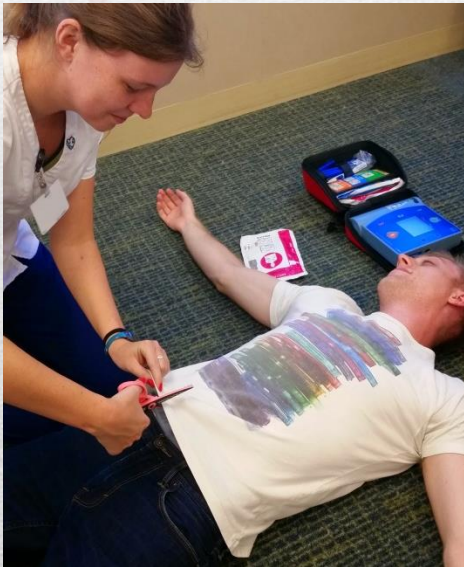

As soon as AED arrives:
1. Press On/Off button
2. Remove clothing from patient's upper body
3. Remove excess hair and moisture from chest where pads will be placed

3






Open defibrillator pads
package



Pull off backing and
place sticky side on
patient's bare skin

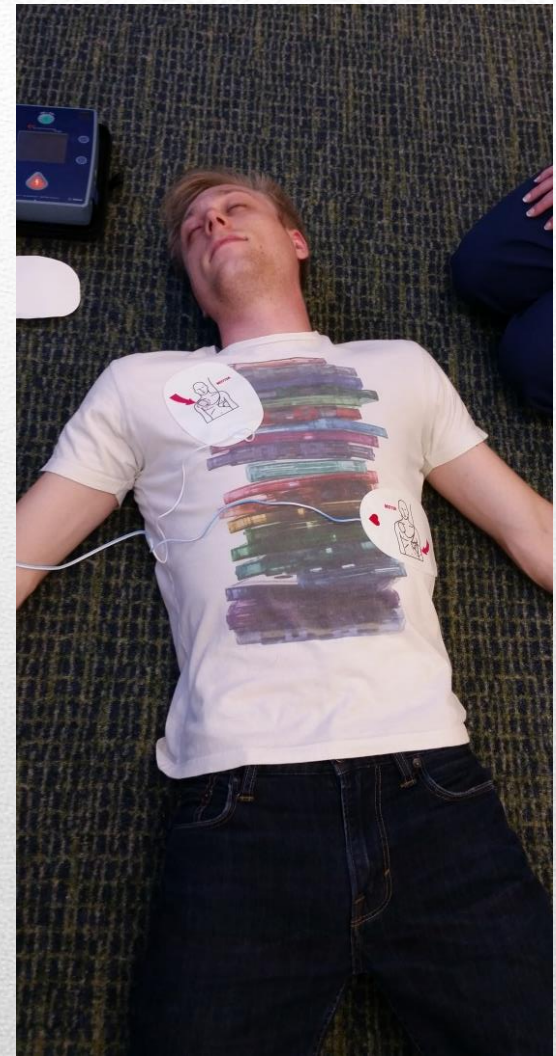


One pad on **left lower**
quadrant below nipple,
and one on **right upper**
quadrant above nipple

If patient appears to be under 8 years of age or 55 pounds, **use infant/child reduced energy pads**, if available.

Place one pad over breastbone and one pad directly opposite of pad on front of body on the back.



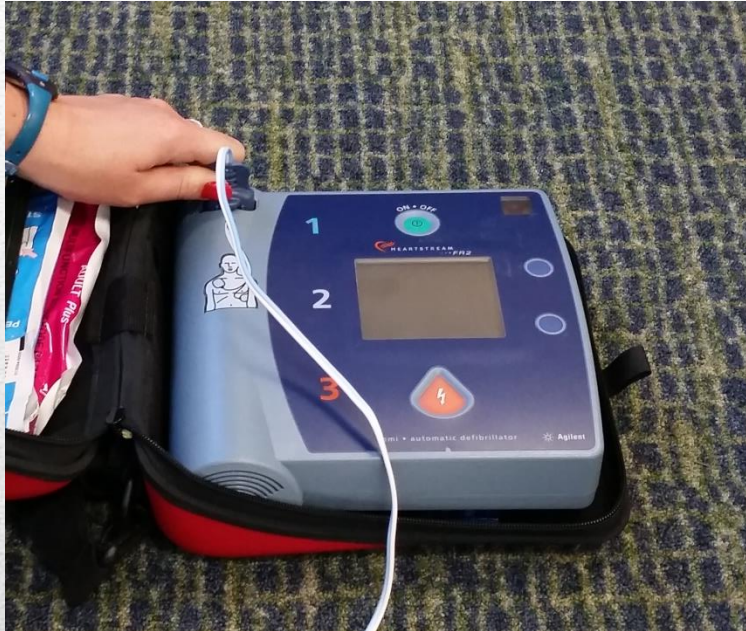


**Insert
defibrillator
connector
firmly into
defibrillator's
connector
socket**

Follow instructions provided by the FR2+ Defibrillator's voice and text prompts.

The defibrillator will analyze the patient's heart rhythm, if pads are placed properly. **DO NOT TOUCH PATIENT WHILE DEFIBRILLATOR IS ANALYZING HEART RHYTHM.**

If **no shock** is advised, the FR2+ will prompt you to begin **CPR**.

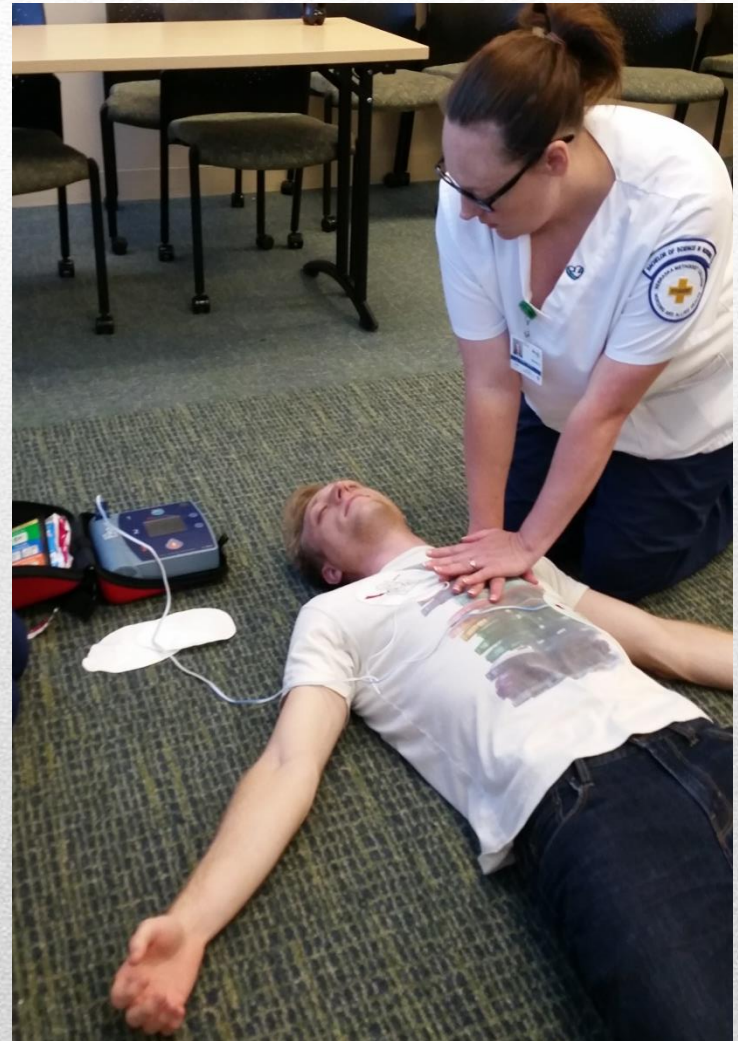



If **shock** is advised, FR+ charges to prepare for shock delivery. Make sure **no one is touching** patient or pads. Press Shock button, a voice prompt tells you the shock was delivered.

4 ways to tell defibrillator is ready to deliver a shock:

- 1. You hear a voice prompt telling you to deliver shock**
- 2. You see the shock button flashing**
- 3. You hear a steady tone**
- 4. You see a text prompt telling you to press the Shock button**





Resume CPR if needed until you hear a prompt telling you to stop or **emergency personnel** has arrived and taken control of the situation.
